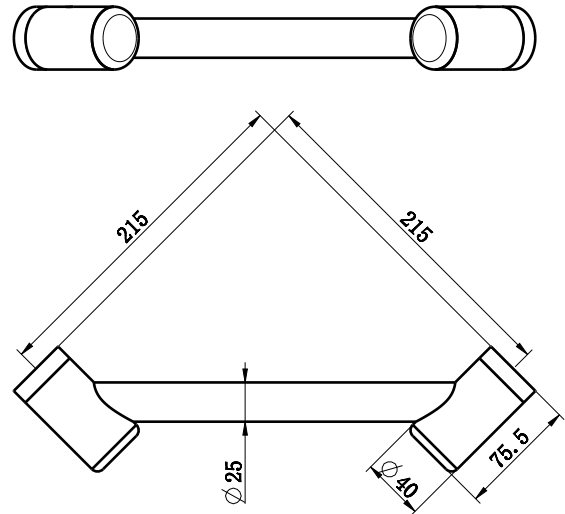








Mecca Care 25MM Footrest Corner Grab Rail 215MM



SKU

	CHROME NR CR2508wCH		MATTE BLACK NR CR2508wMB
	BRUSHED NICKEL NR CR2508wBN		GUN METAL NR CR2508wGM
	BRUSHED GOLD NR CR2508wBG		BRUSHED BRONZE NR CR2508wBZ

Features

1. Australian owned and designed.
2. The corner rail can be mounted at waist height as a grab rail or lower as a footrest.
3. Modular design, the rail to suit your bathroom.
4. 150 kg capacity for all 25mm and 32mm diameter grab rails
5. 25mm diameter rails are suitable for heavy duty towel rails and smaller hands, in non-DDA bathrooms.
6. Gun metal, brushed gold and brushed bronze colours are PVD coated for long term durability.

WARNING:

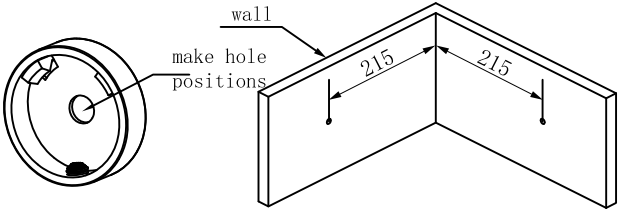
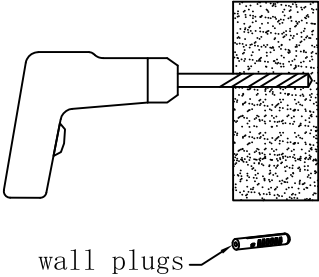
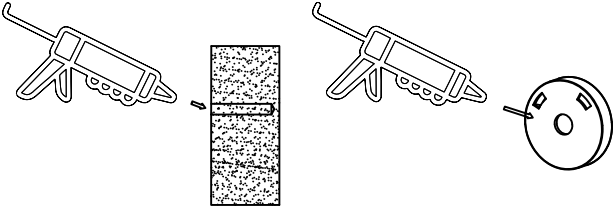

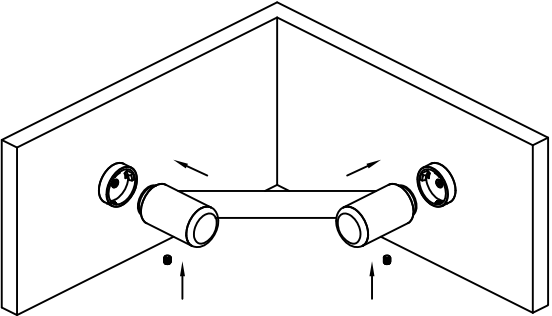
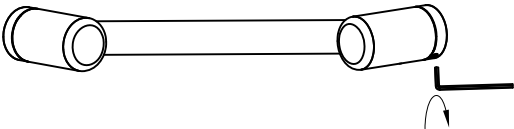
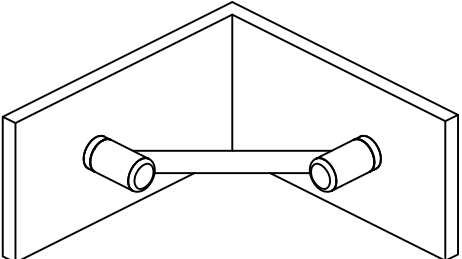
These rails are designed to be mounted onto wall studs or masonry walls. Installation is recommended to be completed by qualified trades person who complies with the AS 1428.1-2009 design for access and mobility standard.

It is recommended rail location is determined by a qualified occupational therapist or trained professional to the AS 1428.1-2009 standard. Correct installation is required to ensure safe, long term operation. Incorrect mounting can result in reduced capacity of the rail which can result in injury to user and damage to the wall.



MECCA CARE
ASSISTED LIVING 25MM

Installation Instruction

	<p>1. Locate wall studs, install into at least one stud. Place wall seal/mounting bracket on wall at correct height over centre of stud and mark hole positions on wall.</p>
	<p>2. Drill pilot holes into wall stud approximately 60mm deep to suit mounting screws (and wall plugs if required).</p> <p>Tip: 4mm hole for screws into timber or 10mm hole for wall plugs.</p>
	<p>3. Apply small amount of silicon to mounting bracket and wall in and around the screw holes.</p>
	<p>4. Install mounting brackets by screwing onto wall.</p> <p>Tip: Use appropriate mounting screws for the wall stud material, supplied screws are for timber wall studs.</p>
	<p>5. Push corner rail onto wall brackets.</p> <p>Tip: Rail length can be cut down to a smaller size, simply use an angle grinder, hacksaw or pipe cutter then deburr the end with a file or grinder and reinsert into the end bracket.</p>
	<p>6. Tighten grub screws with allen key.</p> <p>Tip: If rail rattles, loosen end bracket, twist slightly and tighten.</p>
	<p>7. Finished.</p>